

VitreaView


Quick Reference Guide

NOTE: Images are for physician reference and are not intended for creating the primary image report.

From the Patient's Chart


Radiology images will now be available via the following link under Chart Review OR Results Review:

- Select exam from Imaging Tab in Chart Review or the paper clip for the appropriate exam under Results Review.
- Click on **Show Images** link within Excellian:

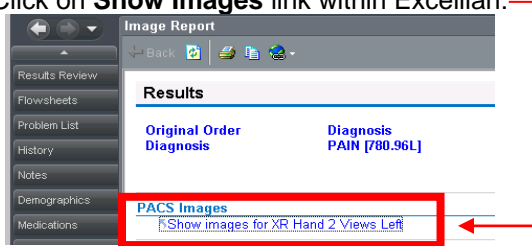

NOTE: For optimal image tool function and speed, open one exam at a time and use the History Tab to view priors.

The following window opens:

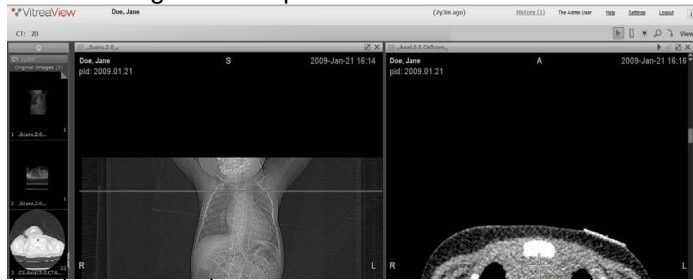


Carousel

Image Viewing Window

**Click and drag studies, series or images from the carousel to the image view window to change.

Navigation Toolbar

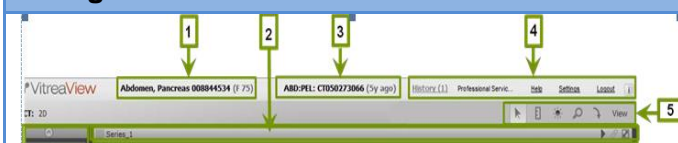

- 1 Patient Box:** Displays the active patient name.
- 2 Image Header:** Displays image identifying information
- 3 Study Box:** Displays the active study
- 4 Global Tools:** Displays global information, including history, help function, and logout.
- 5 Common Tools:** Displays basic tools used in most views.

Global Tools

History (3)	History – displays a count of the prior studies; used to select priors to display
Help	Help – VitreaView Quick Tips

Common Tools


	Arrow – allows you to click on images without changing them
	Ruler – used to measure (right click on measurement to delete) *NOTE: the ruler is only active for CT, MR & NM modalities.
	Sun – use to select presets, reset to return to the original or left click & drag on image to adjust W/L with the mouse
	Magnifying glass – select the magnifying glass icon, left click on the image and drag up to magnify or down to reduce magnification
	Pan – left click and drag to move image on screen
	Rotating Arrow – used to rotate or flip images
View	View – select what shows in the viewer window - View/Reset All will reset all image tools
	Right clicking on the image will display a pie chart of some of the tools above.

Image Tools

	Cine – start/stop, change frame rate
	Auto-link coplanar series
	Change layout to one viewport, second selection returns viewports to original layout
	Close view port

Comparison with Prior Studies

History (3)	Select the History Button to view a list of the patient's prior studies.
--------------------	--

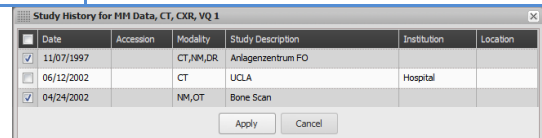

Select exams you would like to view and Apply.

The exams you selected will display in the carousel on the left of the image viewing window.

Change the viewing format to 1 x 2 (see below)

*NOTE: Imaging reports are *not* available in VitreaView, please review imaging reports under the Imaging Tab in Chart Review.

Key Images

Key images will display at the top of the carousel as a series.

Viewing Format

To view images side by side or in four viewports:



Hover over 2D option in the upper left of the VitreaViewer and select the format needed.

Drag and drop images/series from the carousel on left to display images needed for comparison.

Problems: Call the Technology Support Center at 612-262-1900 or 1-800-315-4085 (outside the Twin Cities)
Physicians - Press Option 8